Treatment: Use Ocudox™ twice daily or as recommended by a healthcare professional.

Warnings:

- Do not use the product if tamper resistant seal is broken or missing.
- For external use only. Not for injection or consumption.
- Do not use Ocudox™ in combination with other products without the advice of a healthcare professional.
- In case of deep or serious wounds or if you have any other treatment concerns, consult a healthcare professional.
- If signs of infection occur (increased pain, bleeding, pus) or if the wound does not show signs of healing, consult a healthcare professional.
- Report any adverse reaction to a healthcare professional.

- In addition to using Ocudox™, systemic or local administration antibiotics may be required in some cases.
- Keep out of the reach and sight of children.
- Keep container closed when not using.
- Ocudox™ is harmless in case of accidental contact with nose or ingestion.

Disposal: Ocudox™ is non-toxic to the environment. No special disposal considerations required.

Ingredients: Electrolysed water, Sodium chloride, Sodium phosphate, Sodium sulfate, ancillary substances: Sodium hypochlorite and Hypochlorous acid.

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EN INSTRUCTIONS FOR USE

Indications for use: Ocudox™ is an electrolysed solution intended to aid in the treatment and symptoms of blepharitis. Ocudox™ is used for cleansing the lids, lashes, and periocular skin of debris and microorganisms, which can cause irritation, and ocular surface disease. Can be used for cleansing minor cuts and minor burns.

Handling: Ocudox™ is biocompatible for its indicated use. Non-irritating to human skin, eyes or throat, and is not a skin sensitizer. No special handling precautions are required.

Storage: Ocudox™ should be stored in its original container at 2°C-24°C. Prevent product from freezing. It is non-flammable.

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After use, the bottle should be closed again, before storing.

Stability: Until opened, the stability of Ocudox™ is assured up to expiry date (see bottle for year and month of expiry YYYY MM). Upon opening, note the date on the bottle in the section “Date opened”. Stability of the solution is assured for up to 60 days after opening.

Mode of use: Wash hands prior applied Ocudox™ and remove any make-up or lotions around your eyes. Apply 2 sprays of Ocudox™ on all cotton round pad. Close the eye and using a horizontal motion, gently wipe the base of upper lid lashes at least 3 times. No rinsing is necessary.